

# Data Collection Checklist

## Dentate without Implants

---

This document outlines the requirements for obtaining photo, impression and CBCT data that will be appropriate for planning and ordering an All at Once® Implant Bridge and guides.

This checklist is used for patients who are dentate and do not have any pre-existing implants.

### Patient Preparation:

- Patient consent form completed and signed
- Tooth modifications completed (addition or subtraction)
- Adjust occlusion

### Data Collection Checklist:

#### 1. Patient photos:

- External:
  - Front - lips retracted and teeth apart
  - Front x 3 - smile, biting, and rest
  - Profile x 3 – smile, biting, and rest
  - Above / below
- Internal:
  - Front MI
  - Front open
  - Front protrusive
  - Right and left MI
  - Maxilla (occlusal)
  - Mandible (occlusal)

#### 2. IOS or digital impressions:

- 3 digital files – upper model, lower model, bite model
- Additional scan of RPD if present
- Capture all of the teeth
- Capture as much of the soft tissue as possible
- Must include palate for maxilla
- Capture bite at the planned vertical dimension (use a leaf gauge if necessary)
- Export file format – STL or PLY

#### 3. CBCT:

- Refer to AuDentes® CBCT Scan Protocol – Dentate without Implants
- Export file format – DICOM, uncompressed axial images

